

## AFO Liners and Braces with Ankle Strap

AFO Liners cover the brace – not the leg.

Now the AFO Liner can easily be modified to fit many AFOs with ankle straps.

With AFO Liners on the brace, socks are an option, not a necessity.

Stay cool and trendy in summer foot ware with AFO Liners on the brace.

[www.afoliner.com](http://www.afoliner.com)

email Nona at [customerservice@afoliner.com](mailto:customerservice@afoliner.com)

### **Easy AFO Liner Modification Instruction for Braces with Ankle Straps**

- Tuck ankle strap into the inside of brace and feed the AFO Liner onto the brace. Extend the flap attachment over the top of the brace to attach on back side of brace.
- Set the foot into the brace and with a fine line marker or fabric marker, mark the two areas where the buttonhole type slit will be made. Make one mark where the ankle strap will need to feed through the AFO Liner and on the other side where the ankle strap will attach to the brace. The mark should be made in the center of the base of the strap area and made much shorter than the strap is wide – the fabric will stretch and you want the buttonhole type slit as small as possible. Remove the AFO Liner from the brace.
- Carefully snip the marked areas and turn AFO Liner inside out.
- Very lightly add bonding agent just under the edge of the cut and fold the edge to secure and let set as recommended on bonding agent instructions.
- Turn right side out and add to the brace.

### **Important Information about the Bonding Agent.**

**ONLY USE bonding agent that is Stretchable and Safe for Fabric. Always follow directions for safe usage.** I have used the bonding agent - Mighty Mendit.

